




FIND4NOW™

On farm quick test

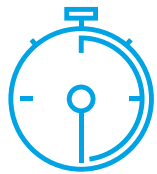

10 MINS

- Results should be read after 10 minutes, not later or earlier than that.



ENSURING RESULTS

- If one of the strips inside the tube dislocates (lifts from the bottom of the tube) use the spatula in the kit to push it down.
- When using the kit, you need to pay attention to whether the strips absorb the liquid. If not, this can be facilitated by tapping the tubes against a firm surface e.g., a table (as described also in the instruction sheet, no. 9).



MIXING

- If testing dry fecal material, homogenizing/mixing the sample with the fluid can be difficult. Mix as well as possible and be extra careful that the strips might clot and not absorb liquid (in this case, the tube can be tapped against a firm surface).

Tips & Tricks

CHR HANSEN

Improving food & health